8sense  
by beurer

## PC 100 Posture Control

<b>DE</b>	Haltungstrainer Gebrauchsanweisung.....	2
<b>EN</b>	Back Trainer Instructions for use.....	28
<b>FR</b>	Coach postural Mode d'emploi.....	52
<b>ES</b>	Detector postural Instrucciones de uso .....	79
<b>IT</b>	Trainer posturale Istruzioni per l'uso .....	105
<b>TR</b>	Postür eğitmeni Kullanım kılavuzu .....	130
<b>RU</b>	Тренажер для осанки Инструкция по применению...	155
<b>PL</b>	Trenażer pozycji Instrukcja obsługi.....	180

## DEUTSCH

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# ENGLISH

**Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.**

**Dear customer,**

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, gentle therapy, blood pressure/diagnosis, weight, massage, beauty, air and baby care.

With kind regards,

Your Beurer team

# Table of contents

1. Getting to know your device .....	32
1.1 Why should you use posture control? .....	32
1.2 The “8sense” app .....	33
2. Intended use .....	34
3. Notes.....	35
4. Device description .....	37
5. Initial use.....	38
6. Use .....	40
7. Transferring data to your smartphone .....	41
8. Replacement parts and wearing parts .....	42
9. Maintenance/cleaning.....	43
10. Frequently asked questions.....	43
11. Disposal .....	45
12. Technical data.....	46
13. Warranty/service .....	48

## Included in delivery

### **WARNING**

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.

1 x Back trainer

1 x USB charging adapter

1 x USB extension cable

1 x Instructions for use

## Signs and symbols

The following symbols appear in these instructions for use and on the device:

	<b>WARNING;</b> warning instruction indicating a risk of injury or damage to health.
	<b>NOTE;</b> note on important information.
	Manufacturer
	Direct current
	This product satisfies the requirements of the applicable European and national directives.
	Dispose of packaging in an environmentally friendly manner

	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	Certification symbol for products that are exported to the Russian Federation and members of the CIS.

## 1. Getting to know your device

### 1.1 Why should you use posture control?

Long-lasting, static positions, such as sitting or standing each day, strain the body, particularly the structures in the back. The posture control helps you to consciously work on your sitting and movement behaviour in order to promote your back health in the long term. Using gentle vibration, the posture control gives you feedback if you have been in static positions for a long time, and thus reminds you to move. You are therefore motivated to adopt active and dynamic sitting and movement behaviour.

The 8sense coaching system can provide you with personalised exercises which compensate for your day's sitting and movement behaviour based on precise analysis of your sitting and movement behaviour during the day.

## 1.2 The “8sense” app

The “8sense” app is available free of charge from the Apple App Store (iOS) or from Google Play (Android™).

### **Effective coaching through an analysis of your posture and movements**

8sense helps you to structure your everyday life in an active manner with plenty of movement in order for you to proactively work on your health. Sitting for too long in one position will trigger haptic feedback to the back in the form of a gentle vibration from the clip.

The posture control also records your micro-movements, changes in position, periods of being upright and periods of reclining.

## System requirements:

iOS  $\geq$  10.0

Android™  $\geq$  5.0

Bluetooth®  $\geq$  4.0

## 2. Intended use

### **WARNING**

The posture control is intended for private use only. The posture control is not a medical device; rather it is a training device. The posture control may only be used for the purpose for which it was designed and in the manner specified in the instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

### 3. Notes

#### **WARNING**

- Avoid contact with sun cream and similar substances as these could damage the plastic parts.
- Keep packaging material and the USB charging adapter away from children. Acute choking hazard!
- Keep the posture control and the USB charging adapter away from water! Never submerge the posture control and the USB charging adapter in water.
- This device is not a medical device. The measured values of the posture control are not suitable for diagnosing, treating, curing or preventing illnesses.
- Only use the device with the supplied accessories.
- Repairs must only be carried out by Customer Services or authorised retailers. Under no circumstances should you open or repair the device yourself, as faultless functionality could no longer be guaranteed thereafter. Failure to comply with this

instruction will void the warranty. Never attempt to repair the device yourself!

- If you are unsure whether the device is suitable for you, please consult your doctor.


### **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
- Protect batteries from excessive heat.

**⚠ Risk of explosion!** Never throw batteries into a fire.

- Fully charge the battery prior to initial use (see the "Charging the battery" section).
- Do not disassemble, open or crush the batteries.
- Only use the USB charging adapter specified in the instructions for use.
- Batteries must be charged correctly prior to use. The instructions from the manufacturer and the specifications in these instructions for use regarding correct charging must be observed at all times.
- Fully charge the battery at least every 6 months.

## 4. Device description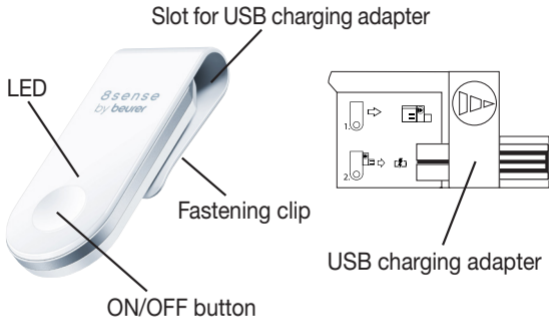

## **LED**

The meaning of the different LED colours and flashing patterns is explained in the “8sense” app.

## **ON/OFF button**

Switching the PC 100 on and off is explained during the introduction process.

## **5. Initial use**



### **Note**

Before using the posture control for the first time, let it charge for a minimum of 1 hour. Proceed as follows:

### **Charging the battery**

1. Once the LED on the posture control lights up red, you have to charge the battery. To do so, plug the USB charging adapter into the side of the posture control. Make sure that the USB charging adapter clicks firmly into place.

2. Connect the USB charging adapter to the USB extension cable, or directly to the USB port on your computer/laptop. Whilst charging, the LED on the posture control lights up orange. When the battery of the posture control is fully charged, the LED goes out. It usually takes approx. 2 hours to fully charge the battery. You can see the current charge status of the posture control in the app if there is a *Bluetooth*<sup>®</sup> connection.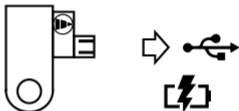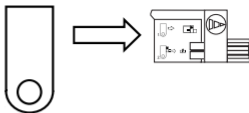

3. Pull the USB charging adapter with the posture control out of the USB port on your computer/laptop.
4. Pull the USB charging adapter out of the side of the posture control. The posture control is now ready for use.

## 6. Use

1. Activate *Bluetooth*<sup>®</sup> on your smartphone.
2. Ensure that your smartphone is connected to the Internet.
3. Download the free “8sense” app from the Apple App Store (iOS) or from Google Play (Android<sup>™</sup>). 
4. Open the “8sense” app and follow the instructions.
5. Switch on the posture control. The LED starts to light up green. If the posture control cannot be switched on, the battery in it is flat and the LED flashes red. Recharge the battery as described in the “Charging the battery” section.
6. Attach the posture control to the back of the neck area of your top as shown (e.g. shirt collar, blouse, T-shirt). Ensure that the long side is facing the inside, and the short side with the clip is on the outside.

7. The posture control is in operation and will now record your body's movements.

## 7. Transferring data to your smartphone

To transfer the recorded data from your posture control to the “8sense” app, proceed as follows:

- Activate *Bluetooth*<sup>®</sup> on your smartphone.
- Open the “8sense” app.

- Keep the smartphone close to the posture control in order to maintain an active *Bluetooth*® connection during data transfer.
- Connect your posture control to the app. Once your smartphone is connected to the posture control via *Bluetooth*®, the data transfer will begin automatically.

## 8. Replacement parts and wearing parts

Replacement parts and wearing parts are available from the corresponding service address (according to the service address list). Please state the corresponding order number.

<b>Designation</b>	<b>Item number and/or order number</b>
USB charging adapter	7250
USB extension cable	164.127

## 9. Maintenance/cleaning

- Clean the posture control after every use using a soft, slightly damp cloth (water or a mild cleaning solution). Dry the posture control using a lint-free cloth. Do not use any abrasive cleaning products.
- Do not submerge the posture control in water or any other fluids and make sure that no fluids can get into the posture control.
- Protect the posture control from knocks, dust, chemicals, drastic changes in temperature, electromagnetic fields and nearby sources of heat (ovens, heaters).

## 10. Frequently asked questions

### **What is the correct way to use the posture control?**

Attach the posture control to the back of the neck area of your top (e.g. shirt collar, blouse, t-shirt), centred on the collar. Ensure

that the long side of the posture control is facing the inside, and the short side is on the outside of the clothing.

### **How do I synchronise my data with the app?**

To transfer the recorded data from your posture control to the “8sense” app, follow the instructions in the “Transferring data to your smartphone” section.

### **The data transfer to the smartphone does not work.**

There could be several reasons for this:

- Check that your smartphone is compatible with the “8sense” app.
- *Bluetooth*® is not activated on your smartphone. Activate *Bluetooth*® in your smartphone settings.
- The battery of the posture control is flat. If the posture control does not switch on, charge it with the USB charging adapter supplied (see the “Charging the battery” section).
- Check that you are using the correct app. Data transfer is only possible with the “8sense” app.

- The posture control is connected to another smartphone. Disconnect the posture control first before connecting it to another smartphone.

### **When should I charge the battery in the posture control?**

Charge the battery in the posture control as soon as the LED on the device flashes red. As soon as the LED flashes red, a maximum battery power of 6 hours remains. In order to continuously record data, never allow the battery to deplete completely.

## **11. Disposal**

Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.

- The codes below are printed on batteries containing harmful substances:  
Pb = Battery contains lead,  
Cd = Battery contains cadmium,  
Hg = Battery contains mercury.

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life.

Dispose of the device at a suitable local collection or recycling point in your country. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## 12. Technical data

Weight	approx. 14 g (incl. charging adapter)
Battery	80 mAh Li-ion polymer
Battery life	max. ~ 40 hours
Permissible operating conditions	0°C - 45°C
Memory capacity	14 days

---

Power supply	5V $\text{---}$ 100 mA
Data transfer via <i>Bluetooth</i> <sup>®</sup> wireless technology	The posture control uses <i>Bluetooth</i> <sup>®</sup> low energy technology, frequency band 2.400–2.480 GHz, the maximum transmission power emitted in the frequency band <20 dBm, compatible with <i>Bluetooth</i> <sup>®</sup> 4.0 smartphones/tablets.
System requirements for the App „8sense“	iOS $\geq$ 10.0, Android™ $\geq$ 5.0, <i>Bluetooth</i> <sup>®</sup> $\geq$ 4.0

---

Technical information is subject to change without notification to allow for updates.

We hereby confirm that this product complies with the European RED Directive 2014/53/EU. The CE Declaration of Conformity for this product can be found under:

<https://www.beurer.com/web/we-landingpages/de/cedeclarationofconformity.php>

## 13. Warranty/service

### Warranty Conditions

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer.**

**The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 2 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;

- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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The *Bluetooth*<sup>®</sup> word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by 8sense GmbH is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc., registered in the U.S. and other countries.

Google Play and the Google Play logo are trademarks of Google LLC.

Android is a trademark of Google LLC.

Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany  
[www.beurer.com](http://www.beurer.com) • [www.beurer-healthguide.com](http://www.beurer-healthguide.com)  
[www.beurer-gesundheitsratgeber.com](http://www.beurer-gesundheitsratgeber.com)