**LANAFORM®**

Downloaded from www.vandenborre.be

# Blood Pressure Monitor with memory

Version 02 of 15/01/2013

Blood pressure

Tension artérielle

Bloeddruk

Figure 1:

Figure 2:

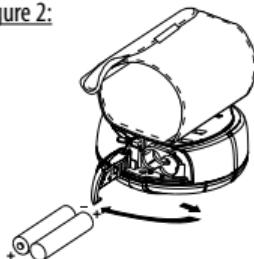
Figure 3:

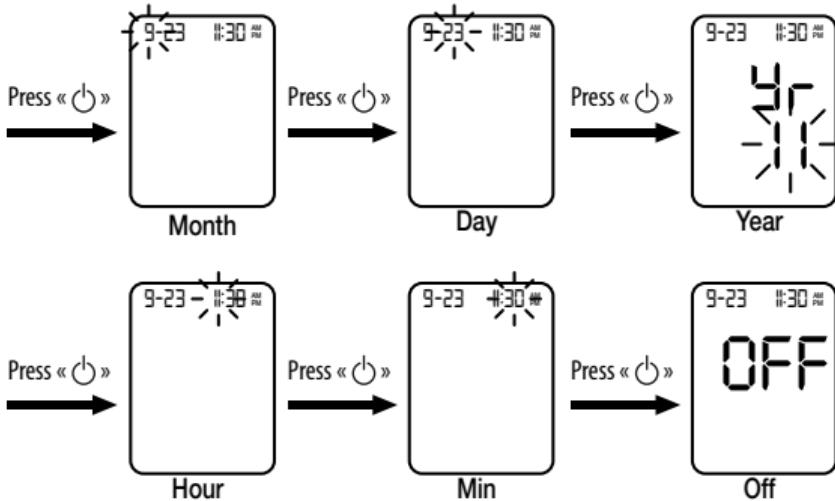
Figure 4:

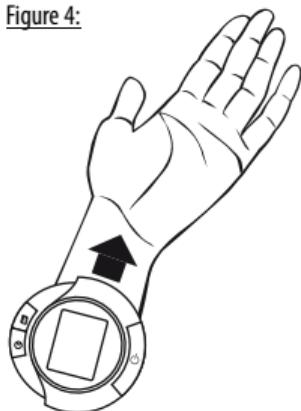
Figure 5:

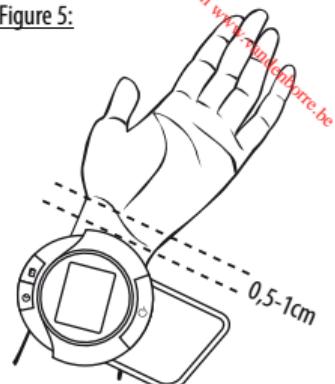
Figure 6:


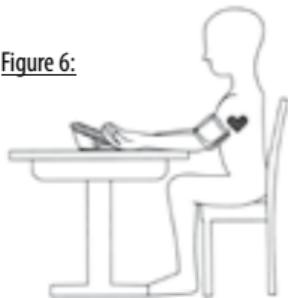
Figure 7:
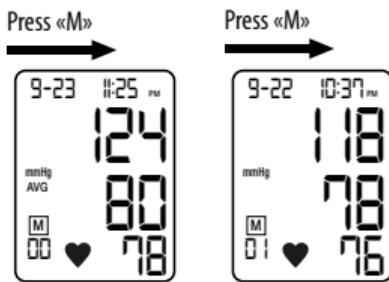
# BLOOD PRESSURE MONITOR WITH MEMORY

Thank you for purchasing the LANAFORM® wrist blood pressure monitor. This monitor uses the oscillometric method to measure human systolic/diastolic blood pressure and heart rate. The monitor detects blood pressure through the wrist artery and converts it into a digital measurement. This monitor does not require a stethoscope, making it very easy to use. With its large LCD screen, you can easily read the various figures. This device is designed for use by adults only.

## **TITLE AND DESCRIPTION OF DISPLAY SYMBOLS ON THE BROCHURE DESCRIBING THE PRODUCT'S FEATURES:**

| Symbol Definition      | Identification | Information provided by the manufacturer |
|------------------------|----------------|--|
| Manufacturer's details |                | LANAFORM SA<br>B-4141 SPRIMONT           |
| Batch number           | LOT            |  |
| BF device              |                |  |

|  |   |  |
|--|---|--|
| Please note: see instructions for use  |  | Please note: consult the enclosed documents.<br>Please read these instructions before using the device. Please also follow these instructions carefully when using the device. |
| Notified body                          | CE 0029   |  |
| Electrical and electronic waste (WEEE) |  |  |

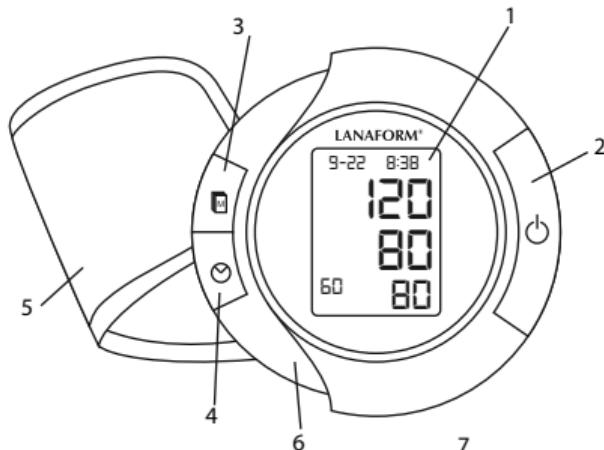
## PLEASE READ ALL THE INSTRUCTIONS BEFORE USING YOUR BLOOD PRESSURE MONITOR, PARTICULARLY THESE FEW BASIC SAFETY INSTRUCTIONS:

- When using this product, please make sure you follow all the instructions listed below. Any deviation from these instructions may cause injury or affect the accuracy of the readings.
- Use only accessories recommended by Lanaform® and supplied with the product. If there is a problem with your device, please contact the supplier or its after-sales department.


- Do not dismantle, repair or convert the monitor.
- Do not expose the monitor to extreme temperatures, high levels of humidity or direct sunlight.
- Always keep the monitor away from heat sources such as stoves, radiators, etc.
- Do not use this device in a room in which aerosols (sprays) are used or in a room in which oxygen is administered.
- Do not use the device in the vicinity of large electromagnetic fields, such as those of cordless or portable telephones.
- Do not treat the device roughly or drop it.
- Do not use the device if it is damaged.
- Do not wash the entire unit in water and do not immerse.
- Pressing the "stop/start" button when the monitor is measuring blood pressure may cause the device to stop running.
- Refrain from talking when the monitor is recording measurements in order to guarantee their accuracy.
- Inaccurate readings may be recorded if users move during the measuring process. It is better to sit down and relax.
- Do not measure blood pressure for 30 minutes after exercising, taking a bath, eating a meal or returning from outdoors. Do not drink alcohol, smoke or take any medication before measuring your blood pressure.

- Downloaded from [www.vandenbussche.be](http://www.vandenbussche.be)
- Relax your wrist and remove any tight clothing or jewellery that could distort the measurements.
  - Place the monitor on your wrist on the side of your heart and place your arm at heart level.
  - Wait at least 15 minutes between two measurements.
  - If you are concerned about your health, consult a doctor before using this device.
  - In any case, results of measurements are ONLY for reference and are not in any way a substitute for a medical opinion.
  - If you feel any pain at all when using this device, stop using it immediately and consult your doctor.
  - Never use this device on any part of the body that is swollen, inflamed or irritated or if you have a rash.
  - This device is not intended for use by people, including children, with limited physical, sensory or mental abilities, or by people lacking experience or knowledge, unless they have been supervised or instructed beforehand on how to use the device by someone responsible for their safety. Children should be supervised to ensure that they do not play with the device.

## DEVICE DESCRIPTION

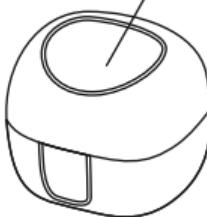
1. LCD screen
2. Stop/start button
3. Memory
4. Time set button
5. Wrist cuff
6. Battery compartment
7. Storage box

## DESCRIPTION OF DISPLAY SYMBOLS

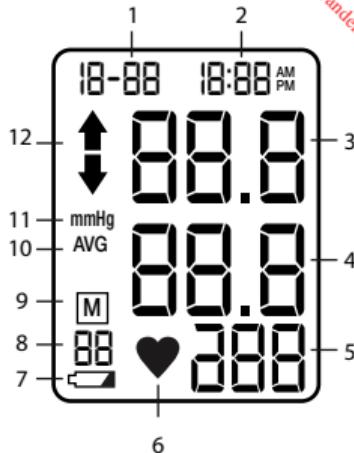1. Date
2. Time
3. Systolic pressure
4. Diastolic pressure
5. Heart rate
6. Pulse symbol
7. Low battery symbol
8. Memory settings
9. Memory symbol
10. Average measurement symbol
11. Unit of measurement symbol
12. Inflation/deflation symbol

## INSERTING/REPLACING THE BATTERIES

Replace the batteries when the low battery symbol « » appears (bottom left-hand corner of the screen) or when the screen does not light up after pressing the start button (see Figure 1).

## To replace the batteries:

1. Open the battery cover in the direction indicated (see Figure 2).
2. Insert two AAA 1.5V alkaline batteries (not included), matching the polarities. Use the symbols "+" and "-" to help you here (see Figure 2).
3. Replace the cover.
4. If the monitor is not to be used for a long time, please remove the two batteries.

## PROCEDURE FOR SETTING THE DATE/TIME

1. Press Time Set button «  » to start the procedure for setting the date and time; the "month" value will blink first.
2. To set this value, please keep pressing the "(M)" button (the value will increase from 01 to 12 each time it is pressed).
3. Press the «  » button again and follow the steps indicated below (Figure 3) to adjust the other values one after the other, i.e. the date, year and time.  
Please press the Time Set button again «  » to confirm and record the values set.

## FIXING THE PRESSURE CUFF

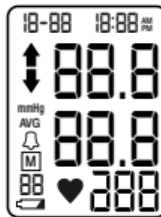
1. Sit down on a seat with your feet on the ground and your arm (heart side) resting on a table. Relax your arm as much as possible.
2. Put the cuff around your wrist "HEART SIDE" with the palm of your hand facing up. The monitor's screen is now facing towards you (Figure 4).
3. Adjust the upper edge of the monitor: it should be between 0.5cm and 1cm below the top of your wrist (Figure 5).
4. Pass the tape around your wrist so that it fits comfortably. Your wrist should not be compressed.

**PLEASE NOTE:** It is natural for blood pressure to vary from one hand to the other, so always take your blood pressure on the same arm to ensure the two readings are comparable.

5. Rest your arm on the storage box and make sure that the monitor is at the same level as your heart (Figure 6).

## HOW TO MEASURE BLOOD PRESSURE


1. Press the Stop/start button.  
- All the display symbols appear for approximately 1 second. This is the initial LCD test.

- When the monitor has completed the necessary preparations prior to measurement, the result of the last reading will be displayed on the screen.

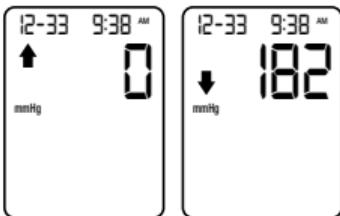- The monitor will begin measuring your blood pressure and the inflation symbol will be displayed during inflation. During deflation, the inflation symbol will turn off and the deflation symbol turn on.

## 2. Measurement results display.

- When measurement is complete, the monitor emits a beep tone. Your systolic and diastolic blood pressure and your heart rate will be displayed automatically on the right-hand side of the LCD screen in compliance with the WHO (World Health Organisation) standards.

**Note:**

Systolic pressure (SYS): When the heart contracts, empties and "pushes" blood into the arteries, heavy pressure is exerted on the artery wall. This is referred to as systolic pressure.

Diastolic pressure (DIA): After the heart contracts, it relaxes. It draws blood back into its chambers, no longer pumping blood into the circulation. The blood pressure is lower during this relaxation phase. This minimum pressure is the diastolic pressure.

- Once the measurement is displayed, it is automatically stored in the memory. When measurement is complete, you simply press the stop/start button again in order to turn your device off. If you forget to turn it off, it will stop automatically after two minutes.

**PLEASE NOTE:** it may take a few seconds for the measurements to disappear.

Inaccurate readings will be recorded if you move when measurements are being taken.

## SETTING THE MEMORY

### Memory recall

- This monitor can store up to 60 readings in its memory. You can therefore create a statement with your doctor of the various measurements taken.
- To turn on the memory, press the "M" button. The first measurement displayed corresponds to an average of all the measurements recorded. The letters AVG will appear on the screen (Figure 7).

- After the average reading is displayed on the screen, you can press the "M" button again to call up the record history from 1 to 60, starting with the most recent records. (Figure 7).
- Press Time Set button « ⏪ » to recall historical records backwards, i.e. from record 60 to record 1.

### **Clearing the memory**

- To clear all records from the memory, please press button "M", holding it down until the screen displays "Clr".
- This function is only operated under power-off mode.

## **MAINTENANCE AND CLEANING**

1. Clean the device carefully using a damp cloth.
2. Do not immerse the device in water. If water enters the device, it could cause damage.
3. Do not use powerful detergents or solvents to clean the device or the cuff.

## TROUBLESHOOTING

| Condition  | Cause  | Corrective Action  |
|--|--|--|
| No display after pressing the stop/start button.<br>Stop/start button. | Batteries inserted incorrectly.                                      | Insert batteries in correct positions.                                     |
|  | Battery voltage is extremely low.                                    | Replace the two batteries with new ones.                                   |
| Replace batteries symbol.  | Battery voltage is extremely low.                                    | Replace the two batteries with new ones.                                   |
|  | Battery function could be reduced in cool conditions.                | Warm up the batteries with your hands or use the device in a warmer place. |
| Life span is reduced.  | This could be due to use of batteries other than alkaline batteries. | Try and use alkaline batteries.  |
| No reading displayed after measurement process.                        | Battery voltage is extremely low.                                    | Replace the two batteries with new ones.                                   |

|   |  |   |
|---|--|---|
| Questionable measurement results.<br>Measurement above or below the user's average measurement. | This could be due to an incorrect position adopted during the measuring process. | Adjust to correct position for measurement:<br><ol style="list-style-type: none"><li>1. Sit down on a seat with your feet on the ground and your arm resting on a table so that the cuff is at the same level as your heart.</li><li>2. Relax your arm and turn the palm of your hand upwards.</li><li>3. Rest your arm on the storage box.</li></ol> |
|   | It is natural for blood pressure to vary from time to time.                      |   |

|  |  |   |
|--|--|---|
| Heart rate is above or below the user's average measurement. | Your arm or body has moved during the measuring process.   | Do not move your arm or your body during the measuring process.   |
|  | Measurement taken immediately after exercise, eating, drinking alcohol or smoking.   | Do not take any measurement immediately after exercise.<br>Do not eat, drink alcohol or smoke before taking measurements. |
| Air is introduced during the the measuring process.          | Operates normally if the user's blood pressure is higher than the initial pressure value, the device automatically pumping to a pressure higher by 40mm Hg each time it is inflated. |   |
|  | The cuff is not positioned correctly.  | Position the cuff correctly by referring to this manual and try again.  |

## ERROR CODES

Err 0: No pulse detected due to incorrect position adopted during the measuring process or due to bulky clothing or jewellery.

Err 1: Serious leakage from the cuff or insufficiently inflated.

Err 2: Systolic pressure (SYS) impossible to obtain:

Err 3: Diastolic pressure impossible to obtain

Err 4: Pressure or pressure outside the legal range impossible to obtain.

Err 5: Deflation is too fast.

Err 6: Deflation is too slow.

Err 7: Systolic pressure outside legal range.

Err 8: Average pressure outside legal range.

Err 9: Cuff pressure is insufficient.

- In all these cases, check that the cuff is positioned correctly. Relax and breathe calmly.  
Try again.

## PRODUCT SPECIFICATIONS

### **Measuring range:**

Blood pressure: 30 to 280mmHG

Heart rate: 40 to 199 beats/min.

### **Accuracy of calibration:**

Blood pressure: +/- 3mmHG

Heart rate: +/- 4%

Power supply: 2 AAA 1.5V alkaline batteries (not included),

### **Conditions of use:**

Temperature: 10 to 40°C

Humidity: 15% to 85% RH (non-condensing)

### **Storage conditions:**

Temperature: -20 to 50°C

Humidity: less than 85% RH (non-condensing)

## ADVICE REGARDING THE DISPOSAL OF WASTE

- The packaging is made up entirely of environmentally-friendly materials that can be taken to your local waste sorting centre to be used as secondary materials. Cardboard can be placed in a paper collection container. Packing film must be taken to your local sorting and recycling centre.
- When you are no longer using the device, dispose of it in an environmentally responsible manner and in compliance with the statutory requirements. Remove the battery beforehand

and dispose of it at a collection point so that it can be recycled.

**Please note :** used batteries should on no account be placed in household refuse!!!

## **WARRANTY**

LANAFORM® guarantees that this product will be free from any defects in material and workmanship for a period of two years from the date it was purchased, other than as specified below.

The LANAFORM® warranty does not cover damage caused by normal wear and tear of this product.

In addition, the warranty on this LANAFORM® product does not cover damage caused as a result of any abusive, inappropriate or incorrect use, accident, attachment of any unauthorised accessory, modification of the product or any other situation of whatever kind, beyond the control of LANAFORM®.

LANAFORM® shall not be held liable for any type of incidental, consequential or special damage.

Any implied warranties of the product's fitness for purpose are limited to a period of two years from the original purchase date on presentation of a copy of the proof of purchase.

Upon receipt, LANAFORM® will repair or replace your device, as appropriate, and return it to you. The warranty may only be used at the LANAFORM® Service Centre. Any maintenance and repair work carried on this product by any person other than the LANAFORM® Service Centre will invalidate this warranty.

## DECLARATION OF CONFORMITY

DECLARATION OF CONFORMITY 0029

Medical device

Category II.a

We, the company LANAFORM, hereby declare that the medical device "Blood Pressure Monitor with memory LA090204" complies with Council Directive 93/42/EEC concerning medical devices and its most recent amendment 2007/47/EEC.

APRAGAZ, 156 chaussée de Vilvoorde, 1120 Brussels (Belgium), notified body no. 0029, has checked this product (according to Annex IV).

## BLOOD PRESSURE MONITOR WITH MEMORY

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